

Now **everyone** can help save a life



HEART START FIRST AID DEFIBRILLATOR

Fast to learn

So simple to operate that a first aider with minimal training can help save the life of a co-worker, friend, member of the family or the public.

Easy to use

HeartStart First Aid analyses the patient's heart rhythm. Calming natural voice instructions coach the first aider through the steps of first aid defibrillation. If required, calming instructive voice prompts will also guide the user in delivering CPR.

Safe

HeartStart First Aid will not allow shock to be applied unless a shockable rhythm has been detected.

Always ready

Comprehensive automatic self-tests check all the HeartStart First Aid's vital functions on a daily, weekly and monthly schedule, so as to ensure that HeartStart First Aid is always ready for emergency life saving.



Laerdal
helping save lives

HeartStart First Aid Defibrillator

Easy to use...



1

CALL 000

- Call 000 if patient is unresponsive and not breathing normally



2

TURN ON HEARTSTART

- Retrieve HeartStart First Aid Defibrillator
- Turn it on by pulling green handle
- Follow the calming and instructive voice prompts



3

APPLY PADS

- Remove pads from cartridge and apply to patient's bare chest



4

HEARTSTART ANALYSES RHYTHM

- Clear voice prompt will inform you whether or not a shockable rhythm has been detected
- Orange light will flash if shock is needed



5

SHOCK

- Press orange button to administer shock
- Do not touch patient



6

CPR

- Perform CPR if necessary
- Press blue button if CPR voice prompts required

LAERDAL AUSTRALIA: 23 EDWARD STREET, OAKLEIGH VICTORIA 3166
HEARTSTART HOTLINE: 1800 DEFIBS (1800 333 427)
Free Fax: 1800 635 835 www.laerdal.com.au

LAERDAL NEW ZEALAND: PO Box 11952 ELLERSLIE AUCKLAND 1542.
Free Call: 0800 523 732 Free Fax: 0800 528 852 www.laerdal.co.nz



Laerdal
helping save lives